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Hergestellt in China

## EN

**Product Contents (Illustration 1):** (1) 1x Ear Wizard, (2) 16x Silicone attachments, (3) 1x Storage Box

### IMPORTANT INFORMATION

Please read the user manual supplied with the Ear Wizard® before using the product. Otherwise, serious injuries can occur.

If you wish to use the Ear Wizard® for children below the age of 12, ask your child's doctor beforehand.

### Ear Wizard® must not be used in the following situations without the supervision of a doctor:

- By people who are suffering from Meniere's disease or any kind of dizziness, vertigo, hearing loss or other ear problems
- By people with ear infections
- By people who wear ear tubes
- By people who have undergone mastoid process surgery
- By people with eardrum perforation at the present time or in the past
- By people with pain of any kind in the ear

### PRECAUTIONARY MEASURES

Do NOT gouge into the ear with the Ear Wizard®. The utensil is NOT suitable for a deep insertion into the auditory canal. The earwax accrues close to the auditory canal inlet. Impacted earwax must be removed by a doctor.

Always rotate the utensil in the direction indicated by the arrow, otherwise serious injuries can occur. If, after use, you wish to reuse a silicone attachment tip, always thoroughly clean and disinfect it beforehand with water and soap. Following frequent use, we recommend discarding and replacing of the attachment tip.

To clean the Ear Wizard wash it with hot water and soap and dry it off before storing away.

### INSTRUCTION MANUAL

The arrow on the Ear Wizard® indicates the direction in which the utensil must be rotated (as seen in illustration A). You should not turn it in any other direction as this can cause serious injuries. To fit an attachment tip, take it in your hand, place it onto the Ear Wizard and rotate it leftwards in order to secure the tip (as seen in illustration B).

The attachment tips have different colours on the underside so that there is no possibility of confusion when the utensil is used by several people.

To remove the tip, take the utensil in your hand and turn the tip rightwards (as seen in illustration C).

### CLEANING

Insert the Ear Wizard® gently and carefully into the ear – Do NOT push the tip too deeply into the inside of the ear. Rotate the utensil solely in the direction of the arrow and do not move during the process in order to avoid injuries (as seen in illustration D+E).

### CHANGING OF ATTACHMENT TIP

If you want to clean the other auditory canal, use a fully clean silicone tip. To remove the already used silicone tip, turn the tip rightwards, remove it and attach a new, clean silicone tip by proceeding in the reverse manner.

### MAINTENANCE

To disinfect immerse used attachment tips in soapy, hot water.

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