
# ZOOM magix®

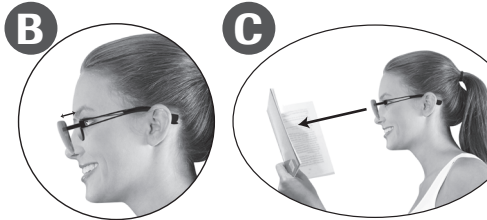DE

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EN

Thoroughly read through the manual and all safety information. Keep these instructions in an accessible place for later reuse.

## SAFETY INFORMATION:

- Not suitable for protection from artificial light sources.
- Not suitable as safety glasses or as part of protective equipment.
- Immediately end use of the magnifiers if you feel any discomfort.
- DO NOT go out wearing ZOOM magix®.
- NOT suitable for the driving of a vehicle or in the participation in any kind of road transport.
- NO solvents or alcohol-based cleaning detergents should be used in the cleaning of the product.
- As with any magnifying glass do not look directly into the sunlight, as this would lead to immediate, irreversible eye damage.

### Warning:

The ZOOM magix® are not corrective spectacles. The magnifiers must not be used as a substitute for corrective spectacles prescribed by an eye specialist or for avoidance of an eye examination. Regularly conducted eye examinations are necessary for the assessment of your eye health.

### APPLICATION: Illustration A

The ZOOM magix® can be worn alone or on top of optical spectacles or sunglasses, reading glasses or any kind of corrective spectacles. The magnifiers must be positioned close to the tip of the nose and NOT in the position that all other types of glasses are usually placed. See Illustration B.

If you wish to wear them on top of corrective or sun glasses, leave space between the ZOOM magix® and the other glasses, so that your eyes can find the correct focal distance (A).

### Finding the focal distance: Illustration C

Hold an object in front of the ZOOM magix® at a distance of approximately 30-40 cm, until you have found the range of the magnifiers that you can comfortably work with.

### CARE AND CLEANING:

For the removal of non-incrusted dirt and finger marks, use the protective and cleaning cover supplied within the product contents or a soft spectacle cleaning cloth. The glasses may also be cleaned with a damp cloth. Make sure, however, that you then immediately dry the glasses afterwards with a soft, dry cloth. If the glasses are very dirty, wash them with water and a mild, non-corrosive cleaning detergent. When cleaning DO NOT exert too much pressure on the glasses, as this may cause scratching to occur.

**Product Contents:** 1 Set of ZOOM magix®, 1 Protective and Cleaning Cover (100% Polyester), 1 User Manual

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CH: MediaShop AG | Industriering 3 | 9491 Ruggell | Liechtenstein
EU: MediaShop GmbH | Schwarzzottstraße 2a | 2620 Neunkirchen | Austria