COMFORTISSE® POSTURE

A

B

C

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EN **Comfortisse® Posture** helps you achieve an upright posture effortlessly.

Warnings:

- Wash Comfortisse® Posture before using it for the first time to reduce any odours. Odours will diminish over time if the product is kept well-ventilated.
- Comfortisse® Posture must fit tightly to the body, but without the support becoming uncomfortable or the blood supply will be impaired. If you find Comfortisse® Posture uncomfortable loosen the fastener belts.
- Make sure you can breathe deeply while wearing Comfortisse® Posture.
- Do not use Comfortisse® Posture while sleeping.
- Stop wearing the garment if your skin breaks out or itches.
- Check the Comfortisse® Posture prior to each time of use for rips or damages, as the article must only be used in faultless condition.

Product Description:

1. Belt
2. Elastic straps

Instructions for Use:

1. Unfasten the belt.
2. Loosen the elastic straps.
3. Put your arms in (A).
4. Adjust the belt to make it as tight and comfortable as possible (B).
5. Fasten the belt end with the aid of the hook velcro fastener.
6. Adjust the elastic straps until you achieve a comfortable, upright posture (C).
7. Pass the ends of the elastic straps through the fasteners.

Material Composition:

Main fabric: 80 % SBR, 20 % Polyester
Elastic Belt: 44 % Elastodiene, 30 % Nylon, 26 % Polyester
Edging: 90 % Polyester, 10 % Elasthan
Belt area around Fastener: 100 % Polyester
Fastener: 60 % Polyester, 40 % Nylon

Cleaning instructions:

Do not expose to the sun | Handwash | Do not bleach | Do not dry in the tumble dryer | Do not iron | Do not dry-clean | Do not wring | Dry in the shade

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CH: MediaShop AG | Industriering 3 | 9491 Ruggell Liechtenstein
EU: MediaShop GmbH | Schwarzottstraße 2a | 2620 Neunkirchen | Austria